

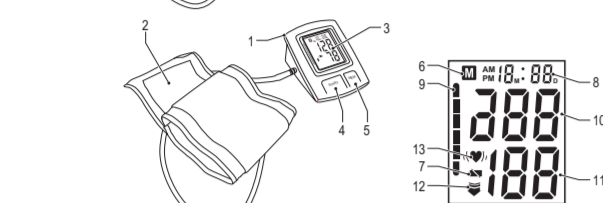
KÖNIG

HC-BLDPRESS22 / HC-BLDPRESS23


Blood pressure monitor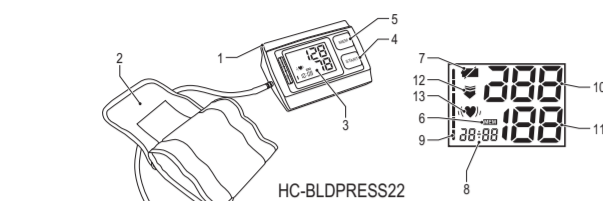

A HC-BLDPRESS22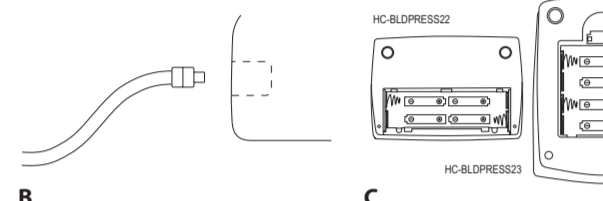

B HC-BLDPRESS23

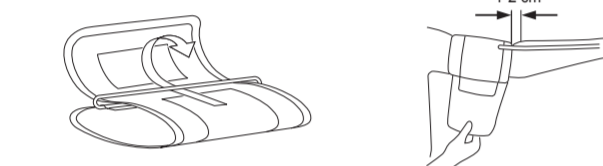D1 D2

E1 E2

Description (fig. A)

The blood pressure monitor is a device used to measure a person's blood pressure. The device measures the systolic blood pressure, the diastolic blood pressure and the pulse rate of an adult individual.

| 1. Monitor | | | | | | | | | | | | | | | | | | | | | | |
|---|---|---------------------------------|--------------------------------|---------------------------------|-------------|------|------|--------|----------|---------|-----------------|-----------|---------|------------------------|-----------|---------|------------------------|-----------|-----------|------------------------|-------|-------|
| 2. Cuff | | | | | | | | | | | | | | | | | | | | | | |
| 3. Display | | | | | | | | | | | | | | | | | | | | | | |
| 4. On/off button | <ul style="list-style-type: none"> Press the button to switch on the device. Press the button again to switch off the device. Clock mode. Press the button to set the date and the time. | | | | | | | | | | | | | | | | | | | | | |
| 5. Memory button | <ul style="list-style-type: none"> Press the button to enter the memory mode. Press the button again to exit the memory mode. Clock mode. Press the button to confirm the date and the time. | | | | | | | | | | | | | | | | | | | | | |
| 6. Memory indicator | <ul style="list-style-type: none"> The indicator comes on when the device is in memory mode. | | | | | | | | | | | | | | | | | | | | | |
| 7. Low-battery indicator | <ul style="list-style-type: none"> The indicator comes on when the batteries are low. | | | | | | | | | | | | | | | | | | | | | |
| 8. Date indicator / Time indicator | <ul style="list-style-type: none"> The indicator alternately shows the date and the time. The indicator shows the blood pressure classification. | | | | | | | | | | | | | | | | | | | | | |
| 9. Blood pressure classification indicator | <table border="1"> <thead> <tr> <th>Blood pressure classification</th> <th>Systolic blood pressure (mmHg)</th> <th>Diastolic blood pressure (mmHg)</th> </tr> </thead> <tbody> <tr> <td>Hypotension</td> <td>< 90</td> <td>> 60</td> </tr> <tr> <td>Normal</td> <td>90 - 119</td> <td>60 - 79</td> </tr> <tr> <td>Prehypertension</td> <td>120 - 139</td> <td>80 - 89</td> </tr> <tr> <td>Hypertension (Stage 1)</td> <td>140 - 159</td> <td>90 - 99</td> </tr> <tr> <td>Hypertension (Stage 2)</td> <td>160 - 179</td> <td>100 - 109</td> </tr> <tr> <td>Hypertensive emergency</td> <td>≥ 180</td> <td>≥ 110</td> </tr> </tbody> </table> | Blood pressure classification | Systolic blood pressure (mmHg) | Diastolic blood pressure (mmHg) | Hypotension | < 90 | > 60 | Normal | 90 - 119 | 60 - 79 | Prehypertension | 120 - 139 | 80 - 89 | Hypertension (Stage 1) | 140 - 159 | 90 - 99 | Hypertension (Stage 2) | 160 - 179 | 100 - 109 | Hypertensive emergency | ≥ 180 | ≥ 110 |
| Blood pressure classification | Systolic blood pressure (mmHg) | Diastolic blood pressure (mmHg) | | | | | | | | | | | | | | | | | | | | |
| Hypotension | < 90 | > 60 | | | | | | | | | | | | | | | | | | | | |
| Normal | 90 - 119 | 60 - 79 | | | | | | | | | | | | | | | | | | | | |
| Prehypertension | 120 - 139 | 80 - 89 | | | | | | | | | | | | | | | | | | | | |
| Hypertension (Stage 1) | 140 - 159 | 90 - 99 | | | | | | | | | | | | | | | | | | | | |
| Hypertension (Stage 2) | 160 - 179 | 100 - 109 | | | | | | | | | | | | | | | | | | | | |
| Hypertensive emergency | ≥ 180 | ≥ 110 | | | | | | | | | | | | | | | | | | | | |
| 10. Systolic blood pressure indicator | <ul style="list-style-type: none"> The indicator shows the systolic blood pressure. | | | | | | | | | | | | | | | | | | | | | |
| 11. Diastolic blood pressure indicator / Pulse rate indicator | <ul style="list-style-type: none"> The indicator alternately shows the diastolic blood pressure and the pulse rate. | | | | | | | | | | | | | | | | | | | | | |
| 12. Ready-to-inflate indicator | <ul style="list-style-type: none"> The indicator comes on when the device is ready to inflate the cuff. | | | | | | | | | | | | | | | | | | | | | |
| 13. Irregular-heartbeat indicator | <ul style="list-style-type: none"> The indicator comes on when the device detects an irregular heartbeat. | | | | | | | | | | | | | | | | | | | | | |

Definitions

- Systolic blood pressure: maximum arterial pressure during contraction of the left ventricle of the heart.
- Diastolic blood pressure: minimum arterial pressure during relaxation and dilatation of the ventricles of the heart when the ventricles fill with blood.
- Pulse rate: number of pulsations in the radial artery per minute.
- Hypotension: low blood pressure.
- Prehypertension: precursor to chronic high blood pressure.
- Hypertension: high blood pressure.
- Hypertensive emergency: high blood pressure with acute impairment of one or more organ systems which can result in irreversible organ damage.

Safety

General safety

- Read the manual carefully before use. Keep the manual for future reference.
- Only use the device for its intended purposes. Do not use the device for other purposes than described in the manual.
- Do not use the device if any part is damaged or defective. If the device is damaged or defective, replace the device immediately.
- The device is designed for adults. Do not use the device on infants or young children. Before use on older children, consult your physician.
- Do not use the device if you suffer from arrhythmia (irregular heartbeats).
- If you are in doubt whether to use the device or not, consult your physician.

- The measurements are only an indication of your blood pressure level. If the device measures a high or low blood pressure level, consult your physician.
- Only use the supplied cuff.
- Measure your blood pressure at approximately the same time each day.
- Before measurement, stay calm and rest for 5 minutes. Relax at least 1 minute between multiple measurements to allow the blood circulation to recover. Too frequent measurements can cause injury due to blood flow interference.
- Do not speak or move during measurement.
- Do not expose the device to water or moisture.
- Do not use the device in areas where interference from other devices can cause inaccurate measurements.
- Do not use the device in a moving vehicle.

Battery safety

- Use only the batteries mentioned in the manual.
- Do not use old and new batteries together.
- Do not use batteries of different types or brands.
- Do not install batteries in reverse polarity.
- Do not short-circuit or disassemble the batteries.
- Do not expose the batteries to water.
- Do not expose the batteries to fire or excessive heat.
- Batteries are prone to leakage when fully discharged. To avoid damage to the product, remove the batteries when leaving the product unattended for longer periods of time.
- If liquid from the batteries comes into contact with skin or clothing, immediately rinse with fresh water.

Use

Installing the batteries (fig. B)

- Refer to the illustration to install the batteries.

Setting the date and the time

If you install the batteries or switch off the device, the display will show date and time alternately. The device is in clock mode.

- Simultaneously press the on/off button and the memory button. A beep will sound.
- Press the memory button to set the month. Press the on/off button to confirm.
- Press the memory button to set the day. Press the on/off button to confirm.
- Press the memory button to set the hour. Press the on/off button to confirm.
- Press the memory button to set the minutes. Press the on/off button to confirm.

Attaching the cuff to the monitor (fig. C)

- Refer to the illustration to attach the cuff to the monitor.

Applying the cuff (fig. D)

- Refer to the illustration to apply the cuff.

Body posture during measurement (fig. E)

- Refer to the illustration for the correct body posture during measurement.

Measuring the blood pressure

- Switch on the device. A beep will sound. The display shows all display items.
- The monitor starts to seek zero pressure.
- The monitor inflates the cuff until there is sufficient cuff pressure.
- The monitor slowly releases air from the cuff and carries out the measurement. The display shows the systolic blood pressure, the diastolic blood pressure and the pulse rate. The results are stored in the memory automatically.
- Switch off the device.

Reading measurements from the memory

- Press the memory button. The display shows the amount of results in the memory, followed by the most recent result.
- Repeatedly press the memory button to show the previous results.

Deleting measurements from the memory

- When the display shows a result, press and hold the memory button for 3 seconds. All results in the memory are deleted after three beeps.

Technical alarm

If the measured blood pressure (systolic or diastolic) is outside the rated range, the display will show "Hi" or "Lo". Consult a physician or check if your operation violated the instructions. The technical alarm is preset in the factory and cannot be adjusted or inactivated. The indication "Hi" or "Lo" will disappear after approximately 8 seconds.

Technical data

| | HC-BLDPRESS22 | HC-BLDPRESS23 |
|--------------------------|--|--|
| Dimensions | 138 x 98 x 48 mm | 136 x 100 x 65 mm |
| Weight (excl. batteries) | 211 g | 236 g |
| Cuff circumference | 22 - 30 cm 30 - 42 cm (option) 42 - 48 cm (option) | 22 - 30 cm 30 - 42 cm (option) 42 - 48 cm (option) |
| Cuff pressure | 0 - 300 mmHg | 0 - 300 mmHg |
| Systolic blood pressure | 60 - 260 mmHg | 60 - 260 mmHg |
| Diastolic blood pressure | 40 - 199 mmHg | 40 - 199 mmHg |
| Pulse rate | 40 - 180 beats/min | 40 - 180 beats/min |
| Accuracy | | |
| Blood pressure | ± 3 mmHg | ± 3 mmHg |
| Pulse rate | ± 5% | ± 5% |
| Memory | 60 measurements (2x) | 60 measurements (2x) |
| Battery | AA (1.5 V) (4x) | AA (1.5 V) (4x) |
| Ambient temperature | | |
| Operation | 5 °C - 40 °C | 5 °C - 40 °C |
| Storage / Transport | -20 °C - 55 °C | -20 °C - 55 °C |
| Relative humidity | | |
| Operation | ≤ 90% | ≤ 90% |
| Storage / Transport | ≤ 90% | ≤ 90% |
| Environmental pressure | 80 - 105 kPa | 80 - 105 kPa |

Error codes

| Error code | Cause | Solution |
|-------------|--|---|
| Er 0 | The pressure system is unstable before measurement. | |
| Er 1 | The device fails to detect the systolic blood pressure. | Do not move and try again. |
| Er 2 | The device fails to detect the diastolic blood pressure. | |
| Er 3 | The pneumatic system is blocked or the cuff is too tight during inflation. | Apply the cuff correctly and try again. |
| Er 4 | The pneumatic system is leaking or the cuff is too loose during inflation. | |
| Er 5 | The cuff pressure is above 300 mmHg. | |
| Er 6 | More than 3 minutes with cuff pressure above 15 mmHg. | Measure again after 5 minutes. If the display still shows the error code, contact the local distributor or the factory. |
| Er 7 | EEPROM accessing error. | |
| Er 8 | Device parameter checking error. | |
| Er A | Pressure sensor parameter error. | |
| No response | Incorrect operation or strong electromagnetic interference. | Remove the batteries, wait for 5 minutes and install the batteries again. |

Cleaning and maintenance

Warning!

- Do not use cleaning solvents or abrasives.
- Do not clean the inside of the device.

- Do not attempt to repair the device. If the device does not operate correctly, replace it with a new device.

- Clean the outside of the device using a soft, damp cloth.
- If you use the device frequently, disinfect the cuff at least twice a week. Apply ethyl alcohol (75-90%) to a soft cloth, wipe the inner side of the cuff with the cloth and allow the cuff to dry naturally.

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